



Xcem Ltd

MERGEBOND (Code: 11420)

INDICATION: Mergebond is a bonding liquid for dentine and enamel bond.

CONTRAINDICATION: In rare cases (delayed and long term) product may cause hypersensitivity or allergy in some people. If such reactions are experienced, discontinue use of the product and refer to physician.

INSTRUCTIONS FOR USE:

Preserve as much enamel as possible and always do a minimal cavity preparation.

Line the deepest part of the cavity with a calcium hydroxide liner. If the cavity is large, make a base with a glass ionomer liner. The base should not extend higher than the dentinal-enamel junction.

Apply a thin layer of Mergebond to all surfaces of cavity, including pins if present and extend beyond the edges of the cavity. Spread Mergebond with fine streams of dry air.

To wet Mergebond, apply compomer.

Light cure for at least 40 seconds, additional curing may be necessary from all aspects of cavity. Mergebond will bond the compomers to tooth.

CAUTION:

Mergebond contains acrylates and polyacrylic acids and other acrylic monomers.

Liquid is flammable, keep away from heat and flames.

Spillages should be clean immediately.

Light sensitive material.

If contact with skin occurs, wash immediately with plenty of water.

If contact with eyes occurs, wash immediately with eye-wash and seek medical advice if necessary.

CONTENTS: See outer pack markings for detail of pack size and contents.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

 Xcem Ltd
13-15 Picardy Road
Belvedere, Kent
DA17 5QH
MADE IN UK
www.xcemdental.com
ISSUE 0319GB



Denvtch Ltd.,
Ground Floor
8-9 Marino Mart,
Fairview, Clontarf,
Dublin 3, D03P590
IRELAND



2803